

USER GUIDE

Xinder Programmer

VERSION 1.0

XINDER TECH LTD


1. Introduction

Xinder Programmer V1.0 now can be used for Swivel Out Seat and Swivel Lift Seat.

Xinder Programmer can be used to update the firmware, download/upload the path file and edit the path program.

1.1 Preparation

To start, the customer needs to prepare a Laptop/Windows Pad and the Program Wire.

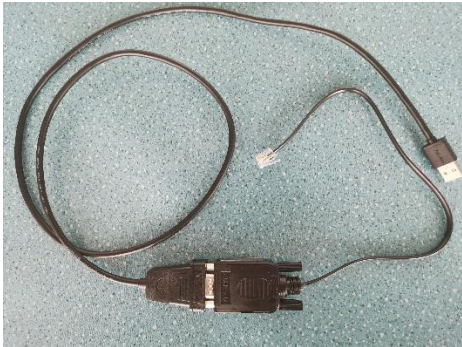

Power up the seat.

Unplug the remote controller and connect the program wire to the seat.



Then Connect the program wire with the computer.

2. Upgrade the Firmware


Make sure the seat connected with the computer.

Try Extract command and check if the Route information is displayed in the Analyze Window.

If yes it means the connection is established otherwise please check the connection first.



Then click Browse button to select the firmware file (.bin file), and click Download button to upgrade. Please note the original firmware version should be higher than 1.3 otherwise it can not be upgraded by this way.

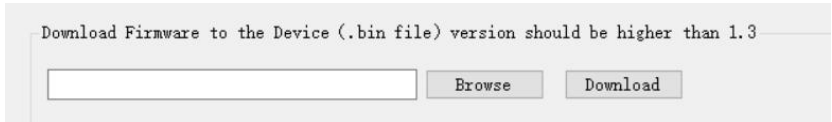

Wait for a few minutes until download process completed (please do not disconnect the device or click any button during the download process)

After download completed, connect the remote controller to the seat again and enter Manual Mode using code “1314” and reset the seat position.


3. Download Path program

The customer can set the seat’s path program by downloading a path file (.txt file)

Click Browse button to choose a path file.

Click Analyze button to show the path details in the analyze window

Click Download button to download the path program to the seat.

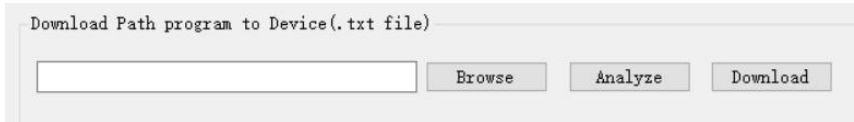

After download completed, connect the remote controller to the seat again and enter Manual Mode using code “1314” and reset the seat position.

4. Extract and Edit/Save the path program

The customer can extract the path program from the seat and edit the program or save the program to a txt file which can be used to download to other seats.

Click Extract button and show the Route information in the Analyze Window.

Click Save as button can save the path program to a txt file.

Click the row in the Analyze window will popup the Edit dialogue which can modify the movement end position.

Click Download button can download the modified path program back to the device.

After download completed, connect the remote controller to the seat again and enter Manual Mode using code “1314” and reset the seat position.

Clear button can clear the information in the Analyze Window and refresh it.

